

Blue to the sky. Orange to the thigh.

How to use EpiPen® and EpiPen Jr® (epinephrine)

Remove the EpiPen® from the carrier tube and follow these 2 simple steps:



- Grasp with orange tip pointing downward
- Remove blue safety cap by pulling straight up – do not bend or twist

- Place the orange tip against the middle of the outer thigh
- Push hard against the leg until you hear a pop, which signals that the injection has started
- Hold in place for 3 full seconds

Built-in needle protection

After injection, the orange cover automatically extends to ensure the needle is never exposed.

After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.

Visit EpiPen.ca

EpiPen® and EpiPen Jr® (epinephrine) are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions. Selection of the appropriate dosage strength is determined according to patient body weight.

EpiPen® and EpiPen Jr® are designed as emergency supportive therapy only. They are not a replacement for subsequent medical or hospital care. After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911. To ensure this product is right for you, always read and follow the label. Please consult the Consumer Information leaflet in your product package for warnings and precautions, side effects, and complete dosing and administration instructions.

Toll free: 1-877-EPIPEN1 (1-877-374-7361)



Trusted for over 35 years.

EpiPen® and EpiPen Jr® are registered trademarks of Mylan Inc., used under permission by BGP Pharma ULC, a Viatris company. VIATRIS and VIATRIS & Design are registered trademarks of Mylan Inc., used under permission by BGP Pharma ULC, a Viatris company. ©2025 Viatris Inc. All rights reserved. CA-EPI-2024-00045E – OC2024



www.EpiPen.ca
Scan here to access how to use EpiPen® video