

Anaphylaxis :

Delaying treatment could be fatal.*

Know what it is

Anaphylaxis is a severe, life-threatening allergic reaction. It can be triggered by certain types of food (like peanuts and shellfish), insect stings, medicine, latex, exercise and unknown causes.

The following symptoms of anaphylaxis can occur within minutes or several hours after exposure to an allergy trigger:

MOUTH: itching, swelling of the lips and/or tongue **THROAT*†:** itching, tightness, closure, hoarseness

SKIN: itching, hives, redness, swelling **GUT:** vomiting, diarrhea, cramps

LUNG*†: shortness of breath, cough, wheeze **HEART*†:** weak pulse, dizziness, fainting

* The EpiPen® auto-injector is not indicated to prevent death associated with anaphylaxis.

† Some symptoms can be life-threatening. ACT FAST!

Know what to do

Epinephrine, the medicine in the EpiPen® auto-injector, is the treatment of choice for severe allergic reactions. If any of the symptoms listed above are exhibited, administer EpiPen® immediately

How to use EpiPen® and EpiPen Jr® (epinephrine):

1

Blue to
the sky



- Grasp with orange tip pointing downward
- **Remove blue safety cap by pulling straight up – do not bend or twist**

2

Orange to
the thigh



- Place the orange tip against the middle of the outer thigh
- **Push hard against the leg until you hear a pop, which signals that the injection has started**
- Hold in place for 3 full seconds

Built-in needle protection

After injection, the orange cover automatically extends to ensure the needle is never exposed.

After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.

Visit EpiPen.ca

EpiPen® and EpiPen Jr® (epinephrine) are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions. Selection of the appropriate dosage strength is determined according to patient body weight.

EpiPen® and EpiPen Jr® are designed as emergency supportive therapy only. They are not a replacement for subsequent medical or hospital care. After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911. To ensure this product is right for you, always read and follow the label. Please consult the Consumer Information leaflet in your product package for warnings and precautions, side effects, and complete dosing and administration instructions.

Toll free: 1-877-EPIPEN1 (1-877-374-7361)



Trusted for over 35 years.

EpiPen® and EpiPen Jr® are registered trademarks of Mylan Inc., used under permission by BGP Pharma ULC, a Viatris company. VIATRIS and VIATRIS & Design are registered trademarks of Mylan Inc., used under permission by BGP Pharma ULC, a Viatris company. ©2025 Viatris Inc. All rights reserved. CA-EPI-2024-00085E - NO2024



www.EpiPen.ca
Scan here to access
how to use EpiPen®
video

