

Blue to the sky. Orange to the thigh.

How to use EpiPen® and EpiPen Jr® Auto-Injectors.

Remove the EpiPen® Auto-Injector from the carrier tube and follow these 2 simple steps:



- Grasp with orange tip pointing downward
- **Remove blue safety cap by pulling straight up**
 - do not bend or twist



- Place the orange tip against the middle of the outer thigh
- **Swing and push the auto-injector firmly into the thigh until it “clicks”**
- Hold in place for 3 full seconds



Built-in needle protection

- After injection, the orange cover automatically extends to ensure the needle is never exposed.

After using EpiPen®, you must seek immediate medical attention or go to the emergency room. For the next 48 hours, you must stay close to a healthcare facility or be able to call 911.



Trusted for over 25 years.

Be informed. Be prepared.

Go to the consumer site **EpiPen.ca** to:

- Sign up for the **FREE** Expiration Reminder Service
- Request additional **FREE** EpiPen® Training Kits and educational brochures for family, friends, and caregivers
- Learn about the signs and symptoms of anaphylaxis
- Register for periodic email updates on new resources

Additional Information and Resources

- Food Allergy Canada: foodallergycanada.ca
- Allergy Asthma Information Association: aaia.ca
- Allergy, Asthma & Immunology Society of Ontario: allergyasthma.on.ca
- Allergies Québec: allergies-alimentaires.org
- The Association of Allergists and Immunologists of Québec: allerg.qc.ca

EpiPen® and EpiPen Jr® (epinephrine) Auto-Injectors are indicated for the emergency treatment of anaphylactic reactions in patients who are determined to be at increased risk for anaphylaxis, including individuals with a history of anaphylactic reactions. Selection of the appropriate dosage strength is determined according to patient body weight.

EpiPen® and EpiPen Jr® Auto-Injectors are designed as emergency supportive therapy only. They are not a replacement for subsequent medical or hospital care. After administration, patients should seek medical attention immediately or go to the emergency room. For the next 48 hours, patients must stay within close proximity to a healthcare facility or where they can call 911. To ensure this product is right for you, always read and follow the label. Please consult the Consumer Information leaflet in your product package for warnings and precautions, side effects, and complete dosing and administration instructions.

© 2017 Pfizer Canada Inc., Kirkland, Quebec H9J 2M5 • Toll free: 1-877-EPIPEN1 (1-877-374-7361)
EpiPen®, EpiPen Jr® are registered trademarks of Mylan, Inc. licensed exclusively to its wholly-owned affiliate,
Mylan Specialty, L.P.; sub-licensee, Pfizer Canada Inc., Kirkland, Quebec H9J 2M5



MEMBER OF
INNOVATIVE MEDICINES CANADA

D000058677



Trusted for over 25 years.